

What are Chewy Tubes?

Chewy Tubes™ are an innovative oral motor device designed to provide a resilient, non-food, chewable surface for practicing biting and chewing skills. **Chewy Tubes™** were originally designed to be used with the **JAW REHABILITATION PROGRAM**, a patented methodology published by Speech Pathology Associates, LLC to develop jaw motion for biting and chewing skills.

Chewy Tubes™ offer a safe, non-toxic surface for developing functional up/down jaw motion. **Chewy Tubes™** do not contain latex, lead, BPA, PVC or phthalates.

Chewy Tubes™ come in a variety of shapes, sizes and textures. Visit www.ChewyTubes.com to learn more about all our **Chewy Tubes™**, **Super Chews™** and **Chewy Q™** products. All **Chewy Tubes™** brand products are made of USFDA approved materials.


Care/Cleaning

Chewy Tubes™ should be inspected before each use and replaced when signs of wear are visible. Clean **Chewy Tubes™** by washing with mild soap and water. Children 3 years of age and younger require adult supervision.

For more information, please contact
SPEECH PATHOLOGY ASSOCIATES, LLC.
P.O. Box 2289
SOUTH PORTLAND, MAINE 04116
TEL 207-741-2443
FAX 207-799-2289
www.chewytubes.com



Chewy Tubes

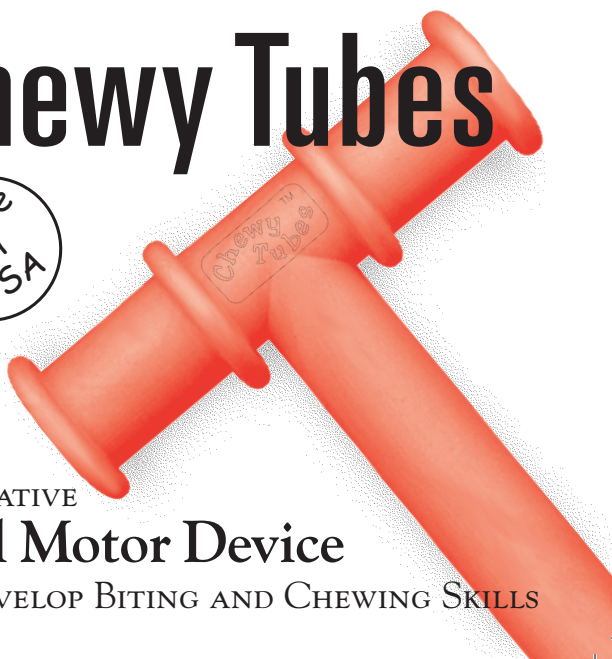

INNOVATIVE

Oral Motor Device

TO DEVELOP BITING AND CHEWING SKILLS

Who Can Use Them?

Chewy Tubes™ offer children and adults a safe and effective tool for practicing biting and chewing skills. Therapists use **Chewy Tubes™** in treatment with individuals to develop oral motor skills and up/down jaw motion for feeding, and to rehabilitate the jaw after surgery, trauma or stroke. Youngsters seeking additional sensory input into the jaw enjoy biting on **Chewy Tubes™**. Children as young as 7 months of age enjoy the tactile input offered by biting on **Chewy Tubes™**. Children with **autism** require adult supervision when using **Chewy Tubes™**.



Chewy Tubes™ are not recommended for use with children under 7 months of age.

How Are They Used?

Chewy Tubes™ provide tactile input at the location of the molar dentition/molar gums as follows:

- 1 Present one **Chewy Tube™** into a side of the mouth.
- 2 Place the bitable stem of the **Chewy Tube™** on the cutting surface of the molar teeth/gums.
- 3 Encourage biting on the **Chewy Tube™** to help improve jaw stability and provide sensory input into the jaw.

Assessment of the jaw is recommended prior to biting on **Chewy Tubes™**. Patients should follow the recommendations of their therapist or caregiver for using **Chewy Tubes™**.



Small and large jaws alike enjoy the tactile input offered by biting on Chewy Tubes™ for fun and therapeutic purposes.

Chewy Tubes™ offer a smooth and resilient surface for developing biting and chewing skills.



To develop biting and chewing skills using Chewy Tubes™, see the **Jaw Rehabilitation Program** at www.chewytubes.com.